


Orthopaedic Pin Packs

Knee Instrumentation Fixation Pins

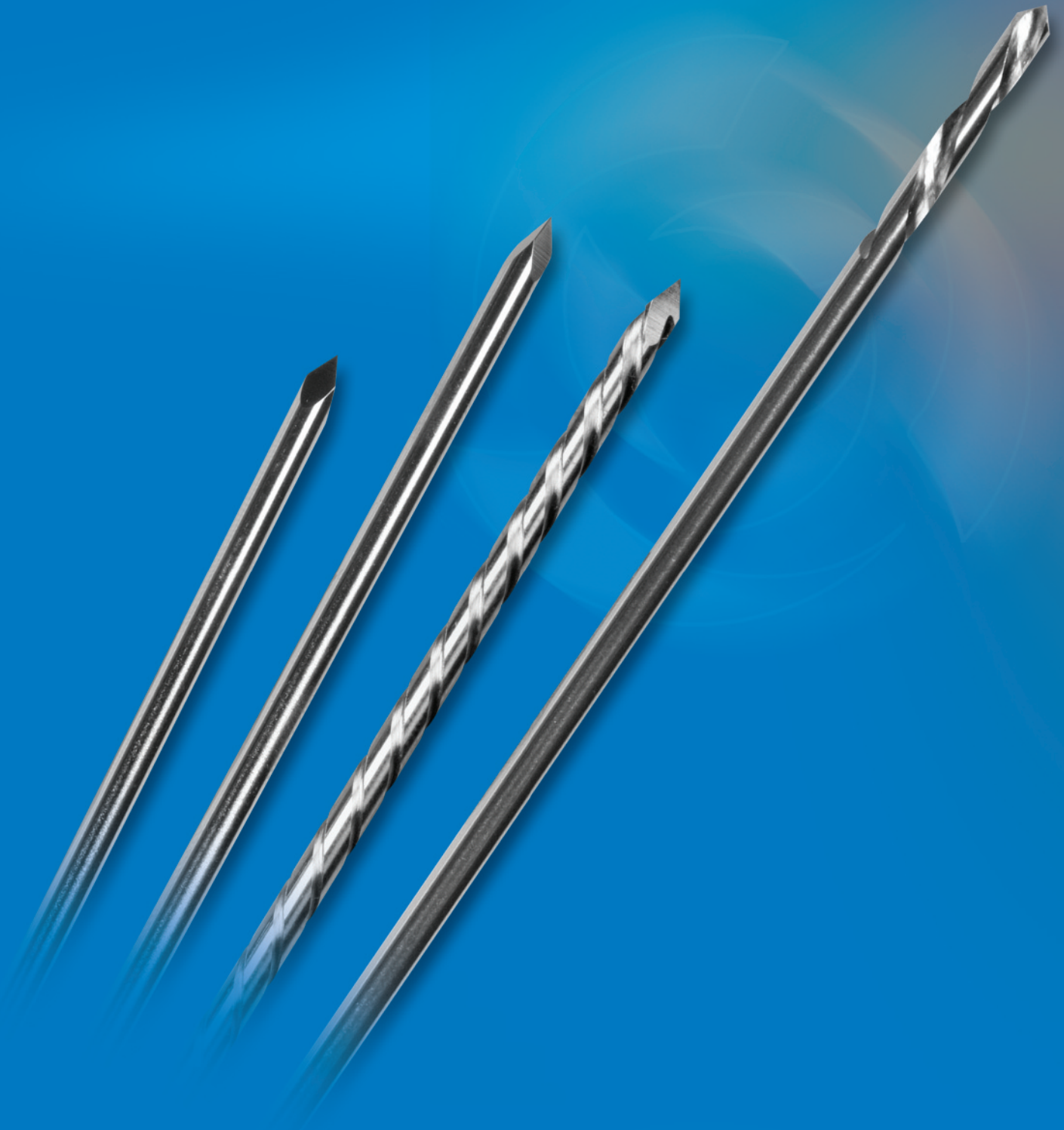

TAVA SURGICAL®
SURGICAL POWER & ACCESSORIES


Orthopaedic Pin Packs - Stryker® Replacements

Fabricated from autoclavable stainless steel.
Features one piece construction.



TAVA Surgical Product #	Stryker® Product #	Product Description	Packaging
BMA-S90008S	<i>7650-2038</i>	1/8" x 3.5" Headless Pin (Square End)	4 Pins/Pack (Sterile)
BMA-S90008	<i>7650-1038</i>	1/8" x 3.5" Headless Pin (Square End)	4 Pins/Pack (Non-Sterile)

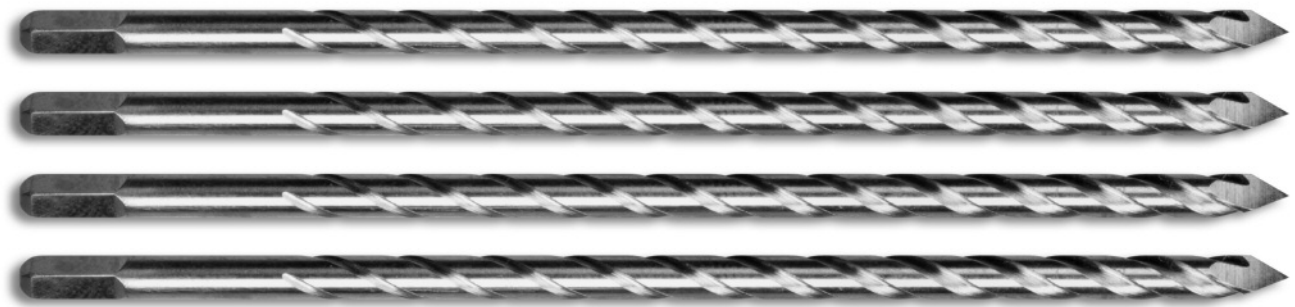

BMA-S90010S	<i>7650-2038A</i>	1/8" x 3.5" Headless Pin, Fluted (Square End)	4 Pins/Pack (Sterile)
BMA-S90010	<i>7650-1038A</i>	1/8" x 3.5" Headless Pin, Fluted (Square End)	4 Pins/Pack (Non-Sterile)





Orthopaedic Pin Packs - Stryker® Replacements

TAVA Surgical Product #	Stryker® Product #	Product Description	Packaging
BMA-S90011S	N/A	1/8" x 3.5" Headless Pin, Fluted (Square End)	5 Pins/Pack (Sterile)
			
BMA-S90012S	6541-4-003	1/8" x 3" Tri Headless Pin (Square End)	4 Pins/Pack (Sterile)
			

Orthopaedic Pin Packs - DePuy® Replacements

Fabricated from autoclavable stainless steel.

Features one piece construction.

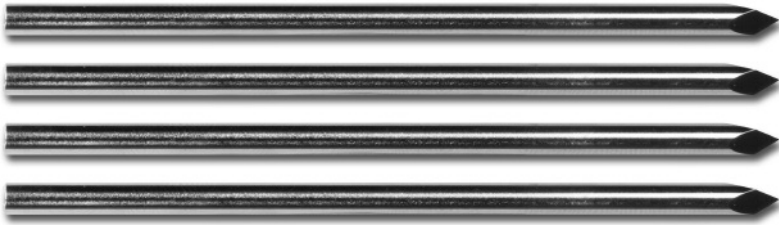TAVA Surgical Product #	DePuy® Product #	Product Description	Packaging
BMA-S90006S	2490-95-000	1/8" x 3" AMK Fixation Pins (Round End)	4 Pins/Pack (Sterile)
			
BMA-S90007S	86-4192	1/8" x 3" PFC Steinman Pin/Drill Pack (Triangular End)	4 Pins & 2 Drills/Pack (Sterile)
			

Sterilization Parameters (Non-Sterile Orthopaedic Pins, Wrapped or Unwrapped)

Sterilization Type	Temperature	Exposure Time	All cycles require at a minimum an 8 minute dry time.
Pre-Vac	270°F (132°C)	4 minutes	<i>Observe AAMI and hospital guidelines for steam sterilization and the sterilizer manufacturer's written instructions for cycle parameters and load configurations.</i>
250°F Gravity	250°F (121°C)	60 minutes total	
270°F Gravity	270°F (132°C)	35 minutes total	



One Brasseler Boulevard • Savannah, Georgia 31419 • 912-921-7575
800-569-6738 Ext. 7050 • 912-921-7578 (fax) • www.TavaSurgical.com

Stryker® and DePuy® are not registered trademarks of TAVA Surgical.